**NOTICE D'INFORMATION DU FABRICANT EPI DE CATEGORIE III**

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References: MECANO-I, MECANO-B-I

Notice n° 91 Indice 1122

**EN****INSTRUCTION SHEET FROM THE MANUFACTURER OF  
CATEGORY III PPE**

This type of glove has been the subject of an EU type (module B) inspection by SATRA Technology Europe Limited, Bracktown Business Park, Clonee, Dublin, D15 YN2P, Ireland. Certified laboratory number: 2777.

The **CE** marking on the glove means that it fulfils the essential health and safety requirements of the European 2016/425 Directive on Personal Protective Equipment.

This product has been designed to provide protection from:

**EN ISO 21420:2020: General requirements**

**EN ISO 374-1:2016+A1:2018 : Protection from hazardous chemicals and micro-organisms**

**EN ISO 374-5:2016 : Performance requirements for micro-organisms risks**

**ISO 1889:2019: Protection for operators handling pesticides and re-**

**EN ISO 374-5:2016: Protection from hazardous chemicals and micro-organisms**

Protection against bacteria and fungi: PASS

Protection from viruses: PASS

**Standard sizes: MECANO-I: size 7 and 9 MECANO B-I: size 8 and 10**

**Dexterity: Level 5**

**Cleaning:** The performance levels are only guaranteed for the glove's palm when new (neither washed, nor regenerated).

**Storage:** Keep the gloves in their original packaging and away from humidity. If unpacked, include a copy of these instructions in the smallest new packaging.

The EU declaration of conformity for the product is available on the web site: [www.rostaing.com](http://www.rostaing.com)

**WARNING: Risks not indicated on this sheet are not covered.**

Composition: nitrile un-powdered glove.

This information does not indicate the actual duration of the protection in the workplace, nor the differentiation between mixtures and pure chemicals.

This product does not cover mechanical and thermal risks.

Chemical resistance was assessed in laboratory conditions using samples taken exclusively from the palm and only concerns the chemical and the sample used for the test. It may be different when used in a mixture.

The test duration corresponds to actual use duration as the permeation test is an accelerated test in which the sample surface is in permanent contact with the

a longer glove should be used  
Single use glove.

This product has been manufactured using a quality system that has been assessed by SGS Fimko Oy, Takomotie 8, FI-00380, Helsinki, Finland. Certified laboratory number: 0598 as per the requirements of PPE regulation (EU) 2016/425 Module D.

For all further information, please contact the manufacturer.

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